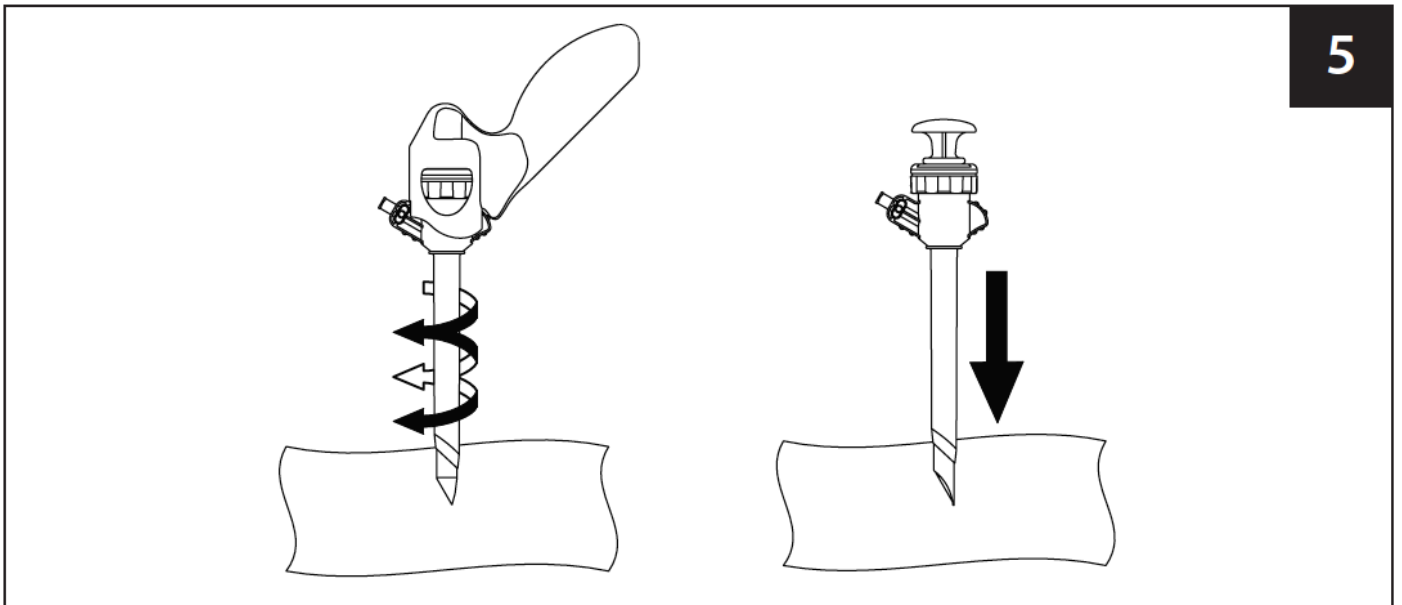
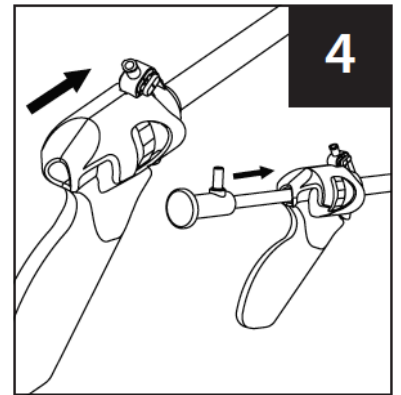
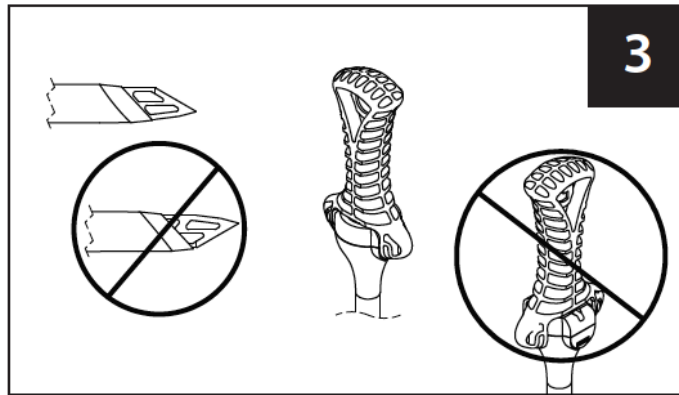
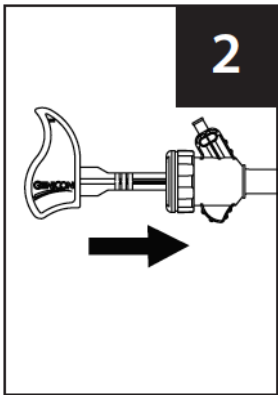
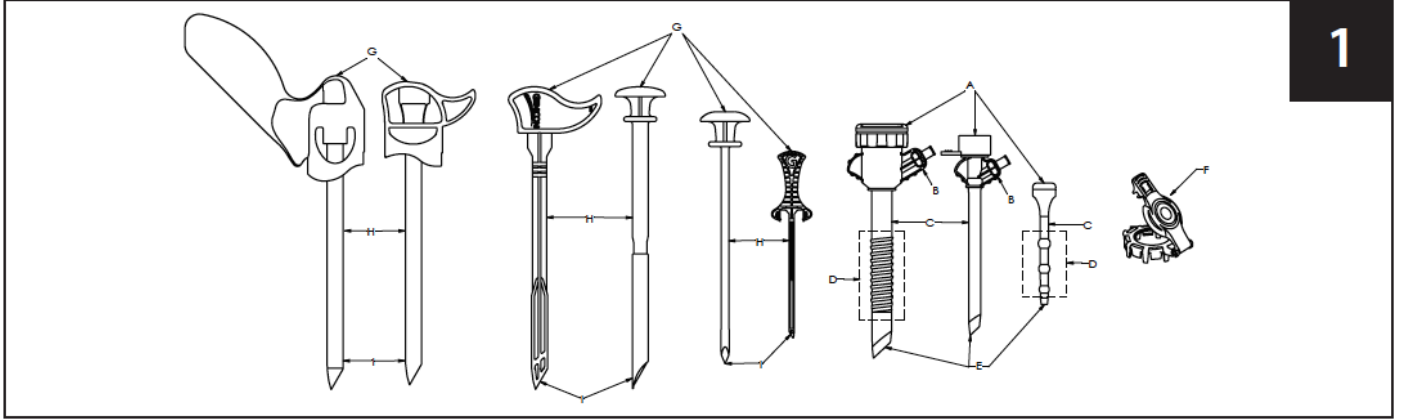
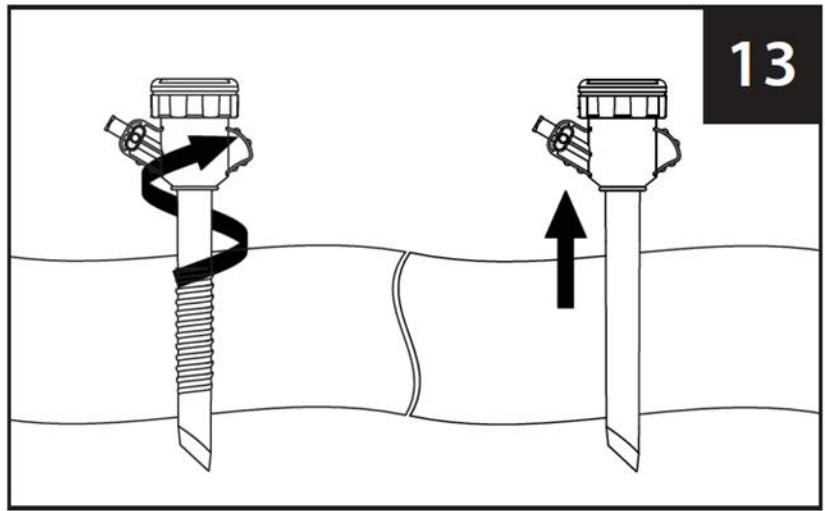
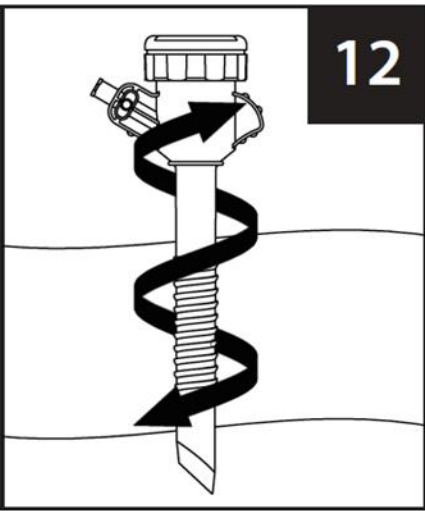
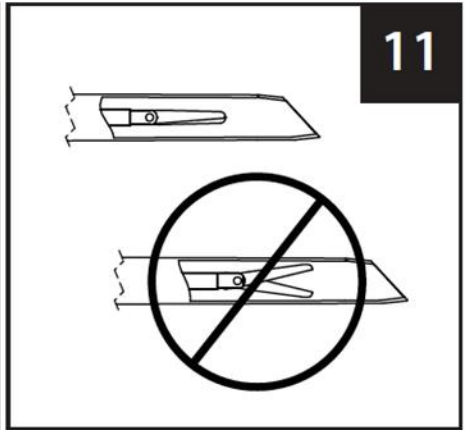
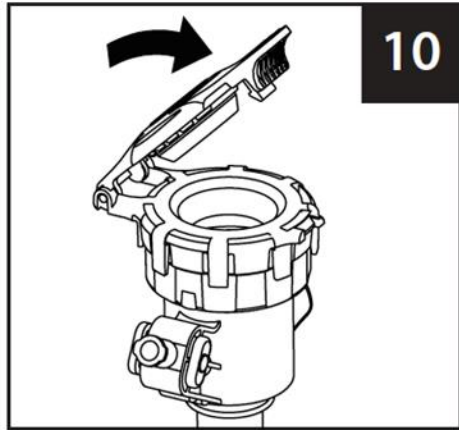
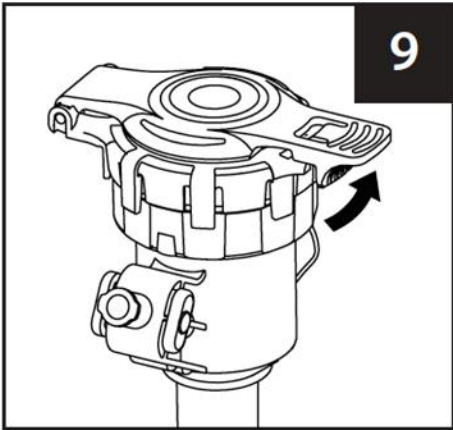
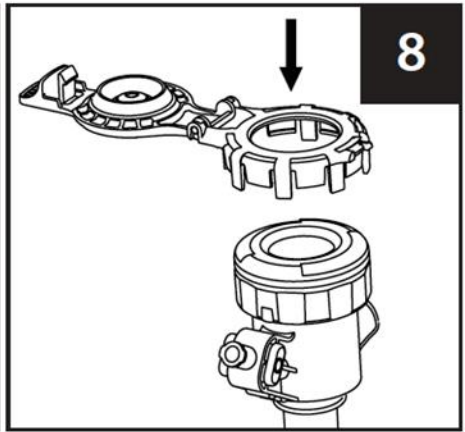
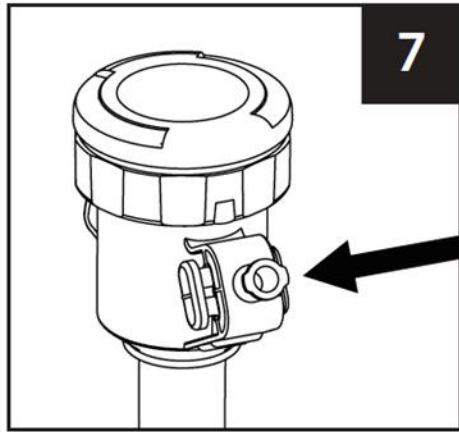
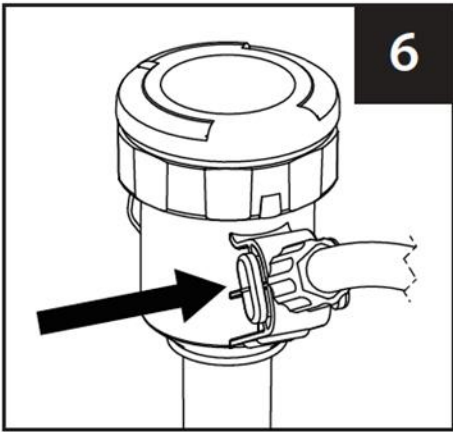
## SINGLE USE TROCAR CANNULA INSTRUCTIONS FOR USE

Including sterile packaged Single-Use Cannula, and sterile packaged Single-Use Pyramidal Tip, Shielded Tip, and Optical and Non-Optical Bladeless Tip

**BEFORE USING THIS PRODUCT, READ THE FOLLOWING INFORMATION THOROUGHLY.**

### IMPORTANT!

1. This booklet is designed to assist using this product. It is not a reference to surgical techniques.
2. This device was designed, tested and manufactured for single use patient use only.
3. Reuse of this device may lead to its failure and subsequent patient injury.
4. Reprocessing and/or re-sterilization of this device may create risk of contamination and patient infection.
5. Do not reuse, reprocess or re-sterilize this device

### DESCRIPTION/INDICATIONS FOR USE

GENICON trocar/cannula systems are intended for surgical insertion into the peritoneal cavity for the purpose of establishing a port of entry for instrumentation and for visualization during laparoscopic for primary and secondary insertions. These devices have applications in gynecologic, general, thoracic, and urologic endoscopic procedures.

### CONTRAINDICATIONS

The use of GENICON laparoscopic and endoscopic products is contraindicated whenever endoscopic surgical techniques are contraindicated for any reason. Contraindications relevant to individual GENICON products are noted in the specific sections.

### WARNINGS AND PRECAUTIONS

1. Endoscopic surgery should be performed only by physicians who are thoroughly trained in endoscopic techniques and failure modes, precautions, and corrective actions in the event of failure.
2. GENICON single-use products are intended for single patient use only – DO NOT RESTERILIZE.
3. Do not use any pre-sterilized products if the packaging sterile barrier is damaged
4. Consult medical literature or country specific regulations for specific techniques, complications, and hazards prior to procedure.
5. Care must be taken when using laparoscopic instrumentation to avoid damage to major vessels and other anatomic structures.
6. Establish and maintain adequate pneumoperitoneum to reduce the risk of injury to internal structures.
7. Properly position the patient and note anatomical landmarks to introduced devices without unintended damage.
8. Do not use excessive force or in a manner not consistent with normal instrumentation use.
9. A thorough understanding of the principles and techniques involved in laparoscopic electro-surgical procedures is essential to avoid shock and burn hazards to both patient and medical personnel, and damage to the device or other surgical instruments.
10. Carry out electro-surgical procedures using only non-combustible gases (CO<sub>2</sub>).
11. Verify that the devices are compatible with other products that will be used in surgery prior to the procedure.
12. Store all products safely in a climate controlled environment and handle with care.
13. All GENICON devices are latex free and sterilized using gamma sterilization, except as noted.
14. Dispose of all used or damaged products using standard hospital practices for biohazard control.
15. Care must be taken when using any trocar/cannula to avoid damage to major vessels and other anatomic structures.
16. The trocar/cannula should be placed under direct endoscopic vision.
17. All instruments should be aligned with the central axis of the cannula when inserting through the seal to minimize the potential of inadvertent damage to the cannula seal. Extra care should be used when inserting angular and /or asymmetrical instruments, such as “J” hooks and clip appliers.

To minimize the risk of such injury, be sure to:

1. Establish and maintain adequate pneumoperitoneum
2. Properly position the patient to help displace organs out of the area of penetration
3. Note important anatomical landmarks
4. Direct the trocar tip away from major vessels and organs
5. Do not use excessive force during insertion
6. Do not use other manufacturer's trocars or cannulae in conjunction with GENICON trocars or cannulae.

### POTENTIAL COMPLICATIONS FOR ALL GENICON® PRODUCTS

Potential complications associated with the use of laparoscopic devices include but are not limited to organ or vessel damage/perforation, hemorrhage, hematoma, trocar site hernia, and septicemia/infection.

### INSTRUCTIONS FOR USE

The following instructions are applicable for all GENICON trocar/cannula systems. Specific instructions are noted after this section.

1. Inspect all components carefully for damage during shipment, handling or after maintenance. **Refer to picture 1**
  - A. Seal/Cap Assembly
  - B. CO<sub>2</sub> Luer Port/Valve
  - C. Cannula Shaft
  - D. Threads/Retaining Rings
  - E. Cannula Tip
  - F. Reducer Valve Assembly (12 mm and larger Cannula only)
  - G. Trocar Handle
  - H. Trocar Shaft
  - I. Trocar Tip
2. Inspect sterile packaging, and verify that sterile barriers are intact. Do not use if package is compromised.
3. Verify that the sizes of all surgical components selected are compatible.
4. Open package using sterile technique and place trocar and cannula (if applicable) on sterile field.
5. Using proper laparoscopy technique, select entry site to ensure the absence of vessels.  
NOTE: DO NOT attach the Reducer Valve Assembly to the cannula prior to port placement.
6. Assemble by gently pressing the trocar tip through the center of the primary valve assembly in the cannula; applying sterile saline to cannula valve will aid in assembly of trocar into the cannula. **Refer to picture 2**
  - 6.1. Shielded and Non-Optical Bladeless Trocars should be oriented so that angle of the blade surface is parallel to the tapered cannula tip if applicable. 3mm Non-Optical Bladeless Trocars should be orientated so that the lateral snap arms on the trocar engage the cannula. **Refer to picture 3**
  - 6.2. The cannula CO<sub>2</sub> port should point upward, and fully seat into the notch on the top of the Optical Bladeless Trocar. A laparoscope may be used with the Optical Bladeless Trocar by inserting the laparoscope into port on the top rear of the trocar. This will aid in visualizing the passage of the trocar through the tissue layers. **Refer to picture 4**
7. Insert the assembly using appropriate laparoscopic technique. **Refer to picture 5**

- 7.1. When using Pyramidal, Shielded or Non-Optical Bladeless Trocars, firmly advance the trocar through the skin and fascia under direct laparoscopic vision. Pyramidal, Shielded and Non-Optical Bladeless trocars typically do not require rotation, although in larger individuals the device may need to be rotated.
- 7.2. For Optical Bladeless Trocars, surgeon should make an incision smaller than the diameter of the cannula. Bladeless Trocar assembly should be rotated with quarter turns back and forth during entry to aid in tissue dissection and insertion.
8. Immediately after insertion of the device, inspect the puncture site for excessive bleeding and assess for organ/vessel damage. Institute appropriate treatment if injury occurs.
9. If necessary, remove trocar from cannula once the cannula has passed through the peritoneum. Smooth or retaining ringed cannulae may require holding during trocar removal to eliminate risk of inadvertent removal from abdomen.
10. If applicable, the insufflation (CO<sub>2</sub>) valve is packaged in the CLOSED position (red slide valve fully depressed). To insufflate the abdomen, attach the CO<sub>2</sub> supply to the luer connection, and press the green colored slide valve fully. **Refer to picture 6** To stop insufflation and close the valve, press the red colored slide valve fully. **Refer to picture 7**
11. To attach/use the Reducer Valve Assembly (12 mm and larger cannula only):
  - 11.1. Remove the Reducer Valve Assembly from the package and open the swing arm by pressing up on the snap arm. **Refer to picture 8**
  - 11.2. Align the arrows on the Reducer Housing with the arrows on the Cannula Cap and press the housing onto the cap until the snap arms are fully seated around the cap. The cannula may require holding during the attachment process. **Refer to picture 8**  
NOTE: Once the reducer is attached to the cap, the trocar may no longer be used with the cannula.
  - 11.3. The reducer valve on the swing arm is designed to seal 4.7 to 5.5 mm instrument. Flip the swing arm onto the cap, pressing down until it snaps into place. **Refer to picture 10**
  - 11.4. The valve in the cap housing is designed to seal 12 mm and larger instruments. Disengage the swing arm from cap by pressing up on the snap arm and flipping it to the side of the cannula. **Refer to picture 9**  
NOTE: Prior to engaging or disengaging the swing arm ensure that all instrumentation is removed from the cannula. Smooth cannula may require holding during the engaging/disengaging process.
12. Instruments should not be opened while inside the cannula. **Refer to picture 11**
13. Threaded cannula depth in the peritoneal cavity may be adjusted by rotating the threads clockwise. **Refer to picture 12**
14. To remove threaded cannula, rotate the threaded cannula counterclockwise. Smooth and retaining ringed cannulae may be removed directly. **Refer to picture 13**

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This product may be covered by one or more U.S. patents and their foreign counter parts. Other Patents may be pending. GENICON and the GENICON logo design are U.S. registered trademarks of GENICON Rights Reserved.

**Disclaimer of Warranties on Re-Processing or Re-Sterilizing of Vendor's Disposable (Single-Patient) Products:** Any reprocessing or re-sterilizing of disposable single patient devices voids Vendor's warranty. Vendor assumes no liability for reprocessed or re-sterilized products. Vendor warrants that reasonable care has been used in the manufacture of these disposable single patient devices and that they are free from defects in workmanship or materials at the time of shipment from Vendor. Vendor's sole obligation shall therefore be to repair or replace any device which it determines was defective at the time of shipment. Because no product is completely effective under all circumstances, and because the actual use and handling of these devices are beyond Vendor's control, Vendor cannot warrant for a good effect or against a bad effect in the application and use of these disposable single patient devices. The Purchaser therefore assumes all liability resulting from re-sterilization or re-processing of these disposable single patient products. Vendor therefore gives no warranty of merchantability or fitness for a particular purpose. Vendor will not be liable for any direct, indirect, incidental or consequential damages resulting from or related to a disposable product that has been re-sterilized or re-processed. Vendor shall not be liable for incidental or consequential loss, damage or expense resulting from the use or application of these said products. This warranty is in lieu of all other warranties, whether implied, express, oral or written, and no individual or entity has the authority to vary the terms of this warranty.

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